

VITA ADIVA® FULL-ADHESIVE LUTING



For feldspar-, glas- and hybrid ceramic restorations
Für Feldspat-, Glas- und Hybridkeramik-Restaurationen

1. Tooth Conditioning

2. Restoration Conditioning

3. Luting

1 2

3 4

5 6

Tooth Conditioning
Konditionierung Zahn

1.

1

VITA ADIVA®
TOOTH-ETCH

1a
1b

30s

20s

20s

20s



2

VITA ADIVA®
T-BOND I + II

1:1

5-10s

massage in

30s

gentle

15s

20s

massage in

30s

gentle

15s

20s

For „Restoration Conditioning“/ „Luting“ please turn over!
Für „Konditionierung Restauration“/ „Befestigung“ bitte wenden!

VITA ADIVA® FULL-ADHESIVE LUTING



1. Tooth Conditioning

2. Restoration Conditioning

3. Luting

For feldspar-, glas- and hybrid ceramic restorations
Für Feldspat-, Glas- und Hybridkeramik-Restaurationen

1 2

3 4

5 6

Restoration Conditioning /
Konditionierung Restauration

2.

Luting /
Befestigung

3.

3 VITA ADIVA® CERA-ETCH

ETHANOL 10s

VITABLOCS® VITA ENAMIC® 60s

VITA SUPRINITY® PC 20s

60s

20s

4 VITA ADIVA® C-PRIME

gentle 20s

5 VITA ADIVA® F-CEM

10s

10s

10s

6 VITA ADIVA® OXY-PREVENT

10s

20s

each side
jede Seite

Light-curing
Lichthärtung