
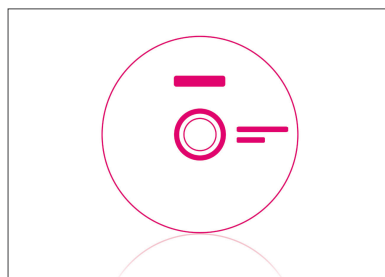
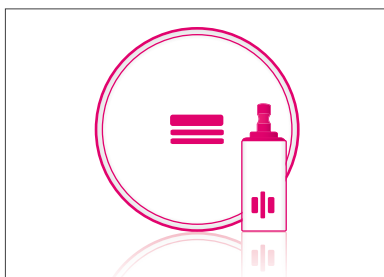
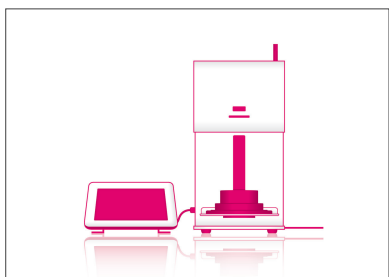
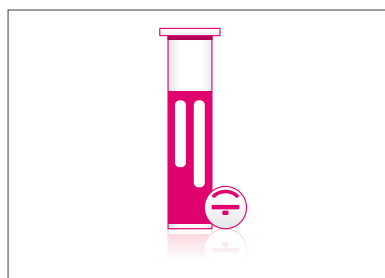
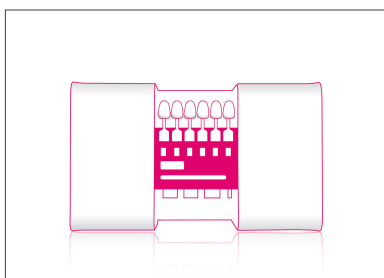


VITA Compendium February 2021

Press technique

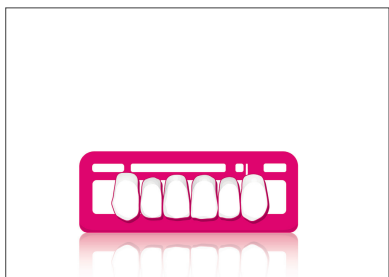

VITA – perfect match.

VITA



VITA – perfect match.

VITA

VITA AMBRIA®



Product description

- Zirconia-reinforced lithium disilicate press ceramic system for highly esthetic and durable results with efficient processing
- The press ceramic system includes press ceramic pellets in two levels of translucency (T/HT) and two sizes (S, L), muffle system, investment material and liquid, as well as press plungers

Features/Benefits

- Brilliant, multi-faceted play of shade and light, thanks to material with natural translucence, opalescence and fluorescence
- Reliably reproduce tooth shades, thanks to press pellets with very good fidelity and perfectly matched system components
- Achieve efficient, highly accurate press results through a minimized reaction layer, as the material and investment material are ideally matched
- Highly durable restorations, thanks to zirconia-reinforced lithium disilicate ceramic with very high stability (> 500 MPa)

Not available in all markets. Official approval pending in some markets.

Indications

For the production of delicate reconstructions, such as veneers and inlays/onlays, as well as highly esthetic, full/partial crowns and three-unit bridges, up to the second premolars.

VITA AMBRIA® kits**VITA AMBRIA® VITA classical A1-D4®****VITA AMBRIA® STARTER KIT VITA classical A1-D4®
LAMSKC1**

Quantity	Content	Material
1	5 pcs	A2-T S
1	5 pcs	A3-T S
4	100 g	VITA AMBRIA® INVEST P (investment powder)
1	100 ml	VITA AMBRIA® INVEST F (investment liquid)
4		VITA AMBRIA® PLUNGER, disposable press plunger Ø 13 mm, 1 piece
2	12 g	VITA LUMEX AC ENAMEL light, fog
1	50 ml	VITA LUMEX AC MODELLING LIQUID
1	5 g	VITA AKZENT Plus GLAZE LT POWDER
1	20 ml	VITA AKZENT Plus POWDER FLUID
1		Firing tray W for crowns and bridges
1		2 Platinum pins (1 anterior and 1 posterior pin)
1	2 pcs.	Fibrous pad firing support
1	60 ml	measuring beaker
1	12 ml	VITA Firing Paste standard package

VITA AMBRIA® products**VITA AMBRIA® VITA classical A1-D4®**

Description	Prod. no.
VITA AMBRIA®, A1-T, L, 3 pcs.	LAM030L0363
VITA AMBRIA®, A2-T, L, 3 pcs.	LAM030L0373
VITA AMBRIA®, A3-T, L, 3 pcs.	LAM030L0383
VITA AMBRIA®, B1-T, L, 3 pcs.	LAM030L0413
VITA AMBRIA®, B2-T, L, 3 pcs.	LAM030L0423
VITA AMBRIA®, C2-T, L, 3 pcs.	LAM030L0463
VITA AMBRIA®, D2-T, L, 3 pcs.	LAM030L0493
VITA AMBRIA®, A1-T, S, 5 pcs.	LAM030S0365
VITA AMBRIA®, A2-T, S, 5 pcs.	LAM030S0375
VITA AMBRIA®, A3-T, S, 5 pcs.	LAM030S0385
VITA AMBRIA®, B1-T, S, 5 pcs.	LAM030S0415
VITA AMBRIA®, B2-T, S, 5 pcs.	LAM030S0425
VITA AMBRIA®, C2-T, S, 5 pcs.	LAM030S0465
VITA AMBRIA®, D2-T, S, 5 pcs.	LAM030S0495
VITA AMBRIA®, A1-HT, L, 3 pcs.	LAM040L0363
VITA AMBRIA®, A2-HT, L, 3 pcs.	LAM040L0373
VITA AMBRIA®, A3-HT, L, 3 pcs.	LAM040L0383
VITA AMBRIA®, B1-HT, L, 3 pcs.	LAM040L0413
VITA AMBRIA®, B2-HT, L, 3 pcs.	LAM040L0423
VITA AMBRIA®, C2-HT, L, 3 pcs.	LAM040L0463
VITA AMBRIA®, D2-HT, L, 3 pcs.	LAM040L0493
VITA AMBRIA®, A1-HT, S, 5 pcs.	LAM040S0365
VITA AMBRIA®, A2-HT, S, 5 pcs.	LAM040S0375
VITA AMBRIA®, A3-HT, S, 5 pcs.	LAM040S0385
VITA AMBRIA®, B1-HT, S, 5 pcs.	LAM040S0415
VITA AMBRIA®, B2-HT, S, 5 pcs.	LAM040S0425
VITA AMBRIA®, C2-HT, S, 5 pcs.	LAM040S0465
VITA AMBRIA®, D2-HT, S, 5 pcs.	LAM040S0495

VITA AMBRIA® VITA SYSTEM 3D-MASTER®

Description	Prod. no.
VITA AMBRIA®, 0M1-T, L, 3 pcs.	LAM030L0023
VITA AMBRIA®, 0M3-T, L, 3 pcs.	LAM030L0043
VITA AMBRIA®, 0M1-T, S, 5 pcs.	LAM030S0025
VITA AMBRIA®, 0M3-T, S, 5 pcs.	LAM030S0045
VITA AMBRIA®, 0M1-HT, L, 3 pcs.	LAM040L0023
VITA AMBRIA®, 0M3-HT, L, 3 pcs.	LAM040L0043
VITA AMBRIA®, 0M1-HT, S, 5 pcs.	LAM040S0025
VITA AMBRIA®, 0M3-HT, S, 5 pcs.	LAM040S0045

VITA PM®9



Product description

VITA PM 9 (Pressable Material) is an "all-in-one" press ceramic and based on the fine-structure veneering ceramic VITA VM 9 which has proved its reliability in millions of restorations. VITA PM 9 covers a wide range of indications. It is suitable for the substructure-free staining and layering technique (inlay/onlay, veneer and crowns) and for overpressing zirconium oxide substructures.

Features/Benefits

Economic fabrication:

The wide indication range (substructure-free staining and layering technique as well as overpressing technique) of VITA PM 9 allows economic and reliable fabrication of various restorations using an identical press program and ensures efficient stock-keeping.

Clinical reliability:

The VITA PM 9 ceramic was developed on the basis of the well-proven fine-structure veneering material VITA VM 9 and hence guarantees utmost clinical reliability.

Natural shade effect:

In particular the highly translucent VITA PM 9 HT pellets offer a convincing chameleon effect to ensure a completely natural shade effect.

Characteristics typical of teeth:

The natural tooth characteristics of the VITA PM 9 ceramic include homogeneous surfaces, high resistance to plaque and enamel-like material behavior.

Indications

Substructure-free staining and layering technique:

Fabrication of inlays, onlays, veneers and crowns.

Overpressing technique:

Overpressing colored and non-colored partially yttrium-stabilized zirconium oxide frameworks

Shades

Opaque-/Translucent-Pellets:

0M1P, 0M2P, 1M1P, 1M2P, 2M1P, 2M2P, 2M3P, 3M1P, 3M2P and 3M3P

High Translucent-Pellets:

Dentine shades: 0M2P, 1M1P, 1M2P, 2M2P, 3M2P Enamel shades: EN0P, EN1P, EN2P, ENDP, ENLP

VITA PM®9 kits

VITA PM®9



VITAPM®9 ACCESSORY KIT EPM9ACCKIT

Quantity	Content	Material
1		VITA PM investment material, 56 x 100 g
1	60 ml	measuring beaker
1	900 ml	VITA PM investment material mixing liquid
1	200 g	VITA PM Investment system, 3 parts (investment ring, sprue base and level gauge)
1	50 pcs.	VITA PM Disposable press plunger



VITAPM®9 HIGH TRANSLUCENT PELLET KIT EPM9HTKIT

Quantity	Content	Material
1	5 pcs.	VITA PM 9 High Translucent, 0M2P-HT
1	5 pcs.	VITA PM 9 High Translucent, 1M1P-HT
1	5 pcs.	VITA PM 9 High Translucent, 1M2P-HT
1	5 pcs.	VITA PM 9 High Translucent, 2M2P-HT
1	5 pcs.	VITA PM 9 High Translucent, 3M2P-HT
1	5 pcs.	VITA PM 9 High Translucent, EN0P-HT
1	5 pcs.	VITA PM 9 High Translucent, EN1P-HT
1	5 pcs.	VITA PM 9 High Translucent, EN2P-HT
1	5 pcs.	VITA PM 9 High Translucent, ENDP-HT
1	5 pcs.	VITA PM 9 High Translucent, ENLP-HT
1		Shade guide for VITA PM 9 high translucent



VITAPM®9 TRANSLUCENT PELLET KIT
EPM9TKIT

Quantity	Content	Material
1	5 pcs.	VITA PM 9 Translucent, 0M1P-T
1	5 pcs.	VITA PM 9 Translucent, 0M2P-T
1	5 pcs.	VITA PM 9 Translucent, 1M1P-T
1	5 pcs.	VITA PM 9 Translucent, 1M2P-T
1	5 pcs.	VITA PM 9 Translucent, 2M1P-T
1	5 pcs.	VITA PM 9 Translucent, 2M2P-T
1	5 pcs.	VITA PM 9 Translucent, 2M3P-T
1	5 pcs.	VITA PM 9 Translucent, 3M1P-T
1	5 pcs.	VITA PM 9 Translucent, 3M2P-T
1	5 pcs.	VITA PM 9 Translucent, 3M3P-T
1		Shade guide for VITA PM 9 translucent

VITA PM®9 products**VITA PM®9**

Description	Prod. no.
VITA PM 9 Opak, 0M1P-O, 5 pcs.	EPM90M1PO
VITA PM 9 Translucent, 0M1P-T, 5 pcs.	EPM90M1PT
VITA PM 9 High Translucent, 0M2P-HT, 5 pcs.	EPM90M2PHT
VITA PM 9 Opak, 0M2P-O, 5 pcs.	EPM90M2PO
VITA PM 9 Translucent, 0M2P-T, 5 pcs.	EPM90M2PT
VITA PM 9 High Translucent, 1M1P-HT, 5 pcs.	EPM91M1PHT
VITA PM 9 Opak, 1M1P-O, 5 pcs.	EPM91M1PO
VITA PM 9 Translucent, 1M1P-T, 5 pcs.	EPM91M1PT
VITA PM 9 High Translucent, 1M2P-HT, 5 pcs.	EPM91M2PHT
VITA PM 9 Opak, 1M2P-O, 5 pcs.	EPM91M2PO
VITA PM 9 Translucent, 1M2P-T, 5 pcs.	EPM91M2PT
VITA PM 9 Opak, 2M1P-O, 5 pcs.	EPM92M1PO
VITA PM 9 Translucent, 2M1P-T, 5 pcs.	EPM92M1PT
VITA PM 9 High Translucent, 2M2P-HT, 5 pcs.	EPM92M2PHT
VITA PM 9 Opak, 2M2P-O, 5 pcs.	EPM92M2PO
VITA PM 9 Translucent, 2M2P-T, 5 pcs.	EPM92M2PT
VITA PM 9 Opak, 2M3P-O, 5 pcs.	EPM92M3PO
VITA PM 9 Translucent, 2M3P-T, 5 pcs.	EPM92M3PT
VITA PM 9 Opak, 3M1P-O, 5 pcs.	EPM93M1PO
VITA PM 9 Translucent, 3M1P-T, 5 pcs.	EPM93M1PT
VITA PM 9 High Translucent, 3M2P-HT, 5 pcs.	EPM93M2PHT
VITA PM 9 Opak, 3M2P-O, 5 pcs.	EPM93M2PO
VITA PM 9 Translucent, 3M2P-T, 5 pcs.	EPM93M2PT
VITA PM 9 Opak, 3M3P-O, 5 pcs.	EPM93M3PO
VITA PM 9 Translucent, 3M3P-T, 5 pcs.	EPM93M3PT
VITA PM 9 High Translucent, EN0P-HT, 5 pcs.	EPM9EN0PHT
VITA PM 9 High Translucent, EN1P-HT, 5 pcs.	EPM9EN1PHT
VITA PM 9 High Translucent, EN2P-HT, 5 pcs.	EPM9EN2PHT
VITA PM 9 High Translucent, ENDP-HT, 5 pcs.	EPM9ENDPHT
VITA PM 9 High Translucent, ENLP-HT, 5 pcs.	EPM9ENLPHT

Accessories for Press Ceramic



Product description

Investment material/liquid

Ideally matched to VITA AMBRIA press pellets, phosphate-bonded and speed-capable investment material with liquid for precise expansion control and flowability.

VITA AMBRIA INVEST

- VITA AMBRIA INVEST P: available as 50 x 100 g pack
- VITA AMBRIA INVEST F: available as 1-liter bottle


Press plunger

Disposable press plunger for simple handling, without timeconsuming blasting and preheating.

- VITA AMBRIA PLUNGER: available as a pack of 50 pieces (diameter 13 mm)

Accessories for Press Ceramic products
Accessories for Press Ceramic

Description		Prod. no.
VITA PM Disposable press plunger, 50 pcs.		E001
VITA PM Investment system, 3 parts (investment ring, sprue base and level gauge), 100 g		E003
VITA PM Investment system, 3 parts (investment ring, sprue base and level gauge), 200 g		E004
Shade guide for VITA PM 9 opaque		E005
Shade guide for VITA PM 9 high translucent		E009
Shade guide for VITA PM 9 translucent		E010
VITA PM investment material mixing liquid, 900 ml		EEF900
VITA PM investment material, 56 x 100 g		EEM100
VITA AMBRIA® INVEST F (investment liquid), 1000 ml		LAMIF1000
VITA AMBRIA® INVEST P (investment powder), 50 x 100 g		LAMIP10050
VITA AMBRIA® PLUNGER (disposable press plunger Ø 13 mm) 50 pieces		LAMP1350

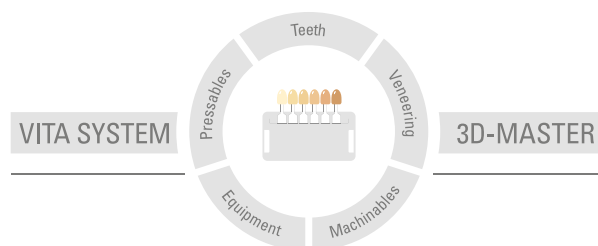

This information issued: 02-2021

Our products were developed specifically for dental use and must be processed according to the instructions for use. The information/illustrations contained herein do not constitute an assurance of properties and are not binding.

VITA

VITA Zahnfabrik H. Rauter GmbH & Co.KG
Postfach 1338 · D-79704 Bad Säckingen · Germany
Tel. +49 (0) 7761/562-0 · Fax +49 (0) 7761/562-299
Hotline: Tel. +49 (0) 7761/562-222 · Fax +49 (0) 7761/562-446
www.vita-zahnfabrik.com · info@vita-zahnfabrik.com
 facebook.com/vita.zahnfabrik