
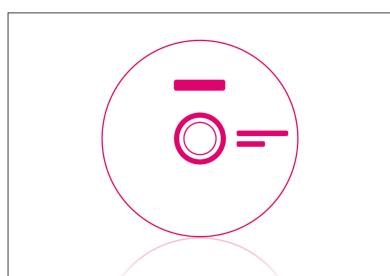
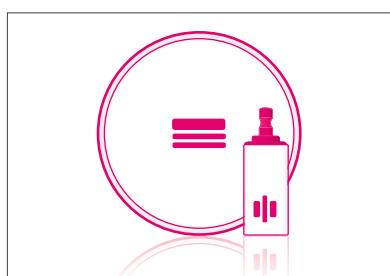
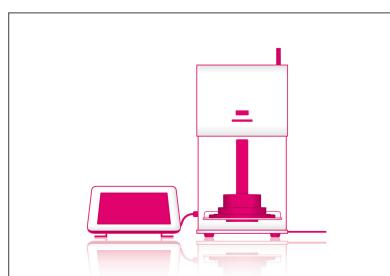
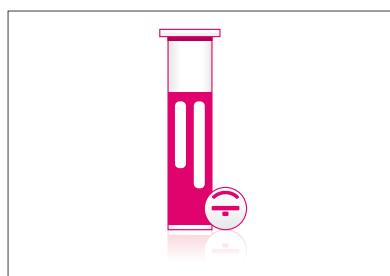


# VITA Compendium September 2021

## Characterization

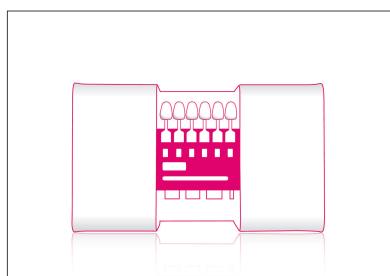

VITA – perfect match.

**VITA**



---

VITA – perfect match.

**VITA**

**VITA AKZENT® Plus**

**Optimizes everything. Effortlessly.**

**Product description**

New areas of application call for new, versatile materials. With VITA AKZENT Plus stains, users can optimize the shade of all types of dental ceramic restorations simply and efficiently, regardless of the CTE of the restoration.

No matter whether internal coloration, surface characterization or fine glazing: with the new VITA AKZENT Plus fluorescent stains, you can adapt your restoration easily and without undue effort.

Depending on the user's preferred/customary method of processing and the relevant area of application, VITA AKZENT Plus stains are available as powders and as a ready-to-use pastes.

The glazing BODY STAINS and GLAZE material are also available as a spray, which, thanks to even application, ensures uniform surface treatment across a large area without any wastage as a result of scatter. The new, special spray head allows precise application to specific areas.

**Features/Benefits**

For all types of dental-ceramic materials, for the layering and press technique and feldspar ceramic blocks such as VITABLOCS to monolithic restorations. It is suitable for all dental ceramic materials and for every CTE.

Available in three different forms depending on the method of application:

**POWDER:**

for unlimited flexibility and cost-effectiveness

**PASTE:**

ready-to-use pastes with uniform consistency and homogeneous pigmentation

**SPRAY:**

ready-to-use, easy-to-apply glaze and finishing agent stains

**VITA AKZENT® Plus kits**

**VITA AKZENT® Plus**



**VITA AKZENT® Plus CHROMA STAINS KIT 3D-MASTER  
BAPKCS3D**

Quantity	Content	Material
5	3 g	VITA AKZENT Plus CHROMA STAINS CHROMA L, CHROMA R, CHROMA IO, CHROMA M2 - CHROMA M3
1	20 ml	VITA AKZENT Plus POWDER FLUID
1		VITA AKZENT Plus CHROMA STAINS, color sample 3D-MASTER



**VITA AKZENT® Plus CHROMA STAINS KIT classical  
BAPKCSC**

Quantity	Content	Material
4	3 g	VITA AKZENT Plus CHROMA STAINS CHROMA A, CHROMA B, CHROMA C, CHROMA D
1	20 ml	VITA AKZENT Plus POWDER FLUID
1		VITA AKZENT Plus CHROMA STAINS, color sample classical



**VITA AKZENT® Plus CHROMA STAINS PASTE KIT 3D-MASTER  
BAPKPCS3D**

Quantity	Content	Material
5	4 g	VITA AKZENT Plus CHROMA STAINS CHROMA L, CHROMA R, CHROMA IO, CHROMA M2 - CHROMA M3
1	15 ml	VITA AKZENT Plus PASTE FLUID
1		VITA AKZENT Plus CHROMA STAINS, color sample 3D-MASTER



**VITA AKZENT® Plus CHROMA STAINS PASTE KIT classical  
BAPKPCSC**

Quantity	Content	Material
4	4 g	VITA AKZENT Plus CHROMA STAINS CHROMA A, CHROMA B, CHROMA C, CHROMA D
1	15 ml	VITA AKZENT Plus PASTE FLUID
1		VITA AKZENT Plus CHROMA STAINS, color sample classical

**VITA AKZENT® Plus PASTE KIT  
BAPKPV1**

Quantity	Content	Material
14	4 g	VITA AKZENT Plus EFFECT STAINS ES01 - ES14
5	4 g	VITA AKZENT Plus BODY STAINS BS01 - BS05
1	4 g	VITA AKZENT Plus GLAZE PASTE
1	4 g	VITA AKZENT Plus GLAZE LT PASTE
1	4 g	VITA AKZENT Plus FINISHING AGENT PASTE
1	15 ml	VITA AKZENT Plus PASTE FLUID
1		Flat brush, for PASTE OPAQUE
1		VITA AKZENT Plus Shade sample disc
1		VITA AKZENT Plus mixing plate, crystal glass (70 x 95 mm)
1		VITA glass spatula for stains
1		Paint brush no. 3/0 synthetic

**VITA AKZENT® PLUS MINI KIT  
BAPSKV1**

Quantity	Content	Material
5	3 g	VITA AKZENT Plus EFFECT STAINS ES01, ES05 - ES06, ES11, ES13
1	5 g	VITA AKZENT Plus GLAZE LT POWDER
1	20 ml	VITA AKZENT Plus POWDER FLUID
1		VITA AKZENT Plus Shade sample disc
1		VITA AKZENT Plus mixing plate, crystal glass (70 x 95 mm)
1		Paint brush no. 3/0 synthetic

**VITA AKZENT® Plus SPRAY KIT  
BAPKSV1**



Quantity	Content	Material
5	75 ml	VITA AKZENT Plus BODY STAINS BS01 - BS05
1	75 ml	VITA AKZENT Plus GLAZE SPRAY
1	75 ml	VITA AKZENT Plus GLAZE LT Spray
1		VITA AKZENT Plus Shade sample disc

**VITA AKZENT® Plus POWDER KIT  
BAPKV1**

Quantity	Content	Material
14	3 g	VITA AKZENT Plus EFFECT STAINS ES01 - ES14
5	3 g	VITA AKZENT Plus BODY STAINS BS01 - BS05
1	5 g	VITA AKZENT Plus GLAZE POWDER
1	5 g	VITA AKZENT Plus GLAZE LT POWDER
1	5 g	VITA AKZENT Plus FINISHING AGENT POWDER
1	20 ml	VITA AKZENT Plus POWDER FLUID
1		VITA AKZENT Plus mixing plate incl. shade sample disc
1		VITA glass spatula for stains
1		Measuring spoon, 0,1 ml / Mixing spatula 2-sided
1		Paint brush no. 3/0 synthetic



**VITA AKZENT® Plus STARTER KIT SPRAY / PASTE  
BAPSKSM**

Quantity	Content	Material
1	75 ml	VITA AKZENT Plus GLAZE LT Spray
1		Brush No. 1 correction brush, red marten
1		Mixing spatula, Acrylic for PASTE OPAQUE
2	4 g	VITA AKZENT Plus CHROMA STAINS CHROMA A, CHROMA B
2	4 g	VITA AKZENT Plus EFFECT STAINS ES01, ES11
1	4 g	VITA AKZENT Plus GLAZE LT PASTE

**VITA AKZENT® Plus products****VITA AKZENT® Plus**

VITA AKZENT Plus BODY STAINS, 3 g (Powder) / 4 g (Paste) / 75 ml (Spray)

Description	Prod. no. 3 g (Powder)	Prod. no. 4 g (Paste)	Prod. no. 75 ml (Spray)
BS01	B505013	B505014	B5050175
BS02	B505023	B505024	B5050275
BS03	B505033	B505034	B5050375
BS04	B505043	B505044	B5050475
BS05	B505053	B505054	B5050575

VITA AKZENT Plus CHROMA STAINS, 3 g (Powder) / 4 g (Paste)

Description	Prod. no. 3 g (Powder)	Prod. no. 4 g (Paste)
CHROMA A	B5050A3	B5050A4
CHROMA B	B5050B3	B5050B4
CHROMA C	B5050C3	B5050C4
CHROMA D	B5050D3	B5050D4
CHROMA IO	B5050I03	B5050I04
CHROMA L	B5050L3	B5050L4
CHROMA M2	B5050M23	B5050M24
CHROMA M3	B5050M33	B5050M34
CHROMA R	B5050R3	B5050R4

Description	Prod. no.
VITA AKZENT Plus GLAZE SPRAY, 75 ml	B5051075
VITA AKZENT Plus GLAZE LT Spray, 75 ml	B5052075
VITA AKZENT Plus FLUOGLAZE LT, 75 ml	B5053075

VITA AKZENT Plus EFFECT STAINS, 3 g (Powder) / 4 g (Paste)

Description	Prod. no. 3 g (Powder)	Prod. no. 4 g (Paste)
ES01	B505513	B505514
ES02	B505523	B505524
ES03	B505533	B505534
ES04	B505543	B505544
ES05	B505553	B505554
ES06	B505563	B505564
ES07	B505573	B505574
ES08	B505583	B505584
ES09	B505593	B505594
ES10	B505603	B505604
ES11	B505613	B505614
ES12	B505623	B505624
ES13	B505633	B505634
ES14	B505643	B505644

Description	Prod. no.
VITA AKZENT Plus GLAZE POWDER, 30 g	B5058130
VITA AKZENT Plus GLAZE PASTE, 4 g	B505814
VITA AKZENT Plus GLAZE POWDER, 5 g	B505815
VITA AKZENT Plus GLAZE LT PASTE, 4 g	B505824
VITA AKZENT Plus GLAZE LT POWDER, 5 g	B505825
VITA AKZENT Plus FINISHING AGENT PASTE, 4 g	B505834
VITA AKZENT Plus FINISHING AGENT POWDER, 5 g	B505835
VITA AKZENT Plus POWDER FLUID, 20 ml	BAPF20
VITA AKZENT Plus PASTE FLUID, 15 ml	BAPPF15

### VITA INTERNO®

For reproducing the shade effects found in the tooth interior



#### Product description

The VITA INTERNO ceramics enable the ceramist to achieve an even more perfect reproduction of in-depth shade effects. Outstanding results can be obtained with INTERNO ceramics, particularly when reproducing colour effects (contrasts) occurring in the incisal area of natural teeth. In special cases the INTERNO ceramics can be used to create an additional impression of depth when space is lacking. Their high fluorescence means shades with increased luminosity and even better light transmission - just as in natural dentition.

VITA INTERNO powders are compatible with **all** VITA veneering ceramics.

#### Features/Benefits

- Compatible with all VITA veneering ceramics
- Can be mixed together
- Stain shades are based on shades naturally found on teeth, so complicated mixing is usually unnecessary
- Do not run; colorfast

**VITA INTERNO® kits****VITA INTERNO®****VITA INTERNO KIT  
BISETV3**

Quantity	Content	Material
14	3 g	VITA INTERNO Int01 - Int14
1	20 ml	VITA INTERNO FLUID
1		VITA INTERNO Shade sample disc
1		VITA INTERNO mixing palette, acrylic (Ø 144 mm, with indentations)
1		VITA glass spatula for stains
1		Paint brush no. 3/0 synthetic

**VITA INTERNO® products****VITA INTERNO®**

VITA INTERNO, 3 g

Description	Prod. no. 3 g
Int01	BI013
Int02	BI023
Int03	BI033
Int04	BI043
Int05	BI053
Int06	BI063
Int07	BI073
Int08	BI083
Int09	BI093
Int10	BI103
Int11	BI113
Int12	BI123
Int13	BI133
Int14	BI143

Description	Prod. no.
VITA INTERNO FLUID, 20 ml	BIF20

## VITA ENAMIC® STAINS

**Reproducing natural shades in a simple and fast manner**



### Product description

The VITA ENAMIC STAINS KIT is comprised of six stains and accessories for the reproduction of natural shade nuances of hybrid ceramic restorations. The stains are bonded to the restoration in a polymerization process. VITA ENAMIC GLAZE, available for surface sealing, increases the durability and brilliance of the shades in the oral environment. Processing is based on a simple step-by-step principle: condition the surface, mix and apply the shades, carry out intermediate polymerization, apply chemical glaze material and perform final polymerization.

### Features/Benefits

#### Fast processing:

Simply apply the VITA ENAMIC stains to the restoration, polymerize and that's it! This way, the shade of VITA ENAMIC restorations can be quickly characterized.

#### High level of individuality:

The anomalies and shade nuances of natural teeth can be individually characterized with the six VITA ENAMIC stains.

#### Simple processing:

The shade intensity of the VITA ENAMIC stains can be perfectly controlled by the ratio of liquid to shade powder. This way homogeneous application of shades is ensured.

### Indications

VITA ENAMIC stains are suitable for the characterization (occlusal/ cervical) of restorations made of VITA ENAMIC hybrid ceramic.

### Shades

Available in six shade nuances: white, corn, brown, khaki, blue and black.

### System compatibility

The following dental and dental-technical polymerization units are recommended for intermediate and final polymerization: Bluephase C8 (Ivoclar Vivadent), Astralis 10 (Ivoclar Vivadent), Valo LED (Ultradent), Speed Labolight (Hager & Werken), Heraflash (Heraeus Kulzer) and Polylux PT (Dreve).

**VITA ENAMIC® STAINS products****VITA ENAMIC® STAINS**

Description		Prod. no.
VITA VM LC CLEANER, 50 ml Cleaning solution		CVMLC50

Description		Prod. no.
VITA Mixing palette black porcelain with lid, 8,5 x 11 cm		C014
Mixing spatula, double-ended		C013
VITA ENAMIC STAIN 1, white, 2 g		EENST012
VITA ENAMIC STAIN 2, corn, 2 g		EENST022
VITA ENAMIC STAIN 3, brown, 2 g		EENST032
VITA ENAMIC STAIN 4, khaki, 2 g		EENST042
VITA ENAMIC STAIN 5, blue, 2 g		EENST052
VITA ENAMIC STAIN 6, black, 2 g		EENST062
Microbrush disposable applicators, 25 pcs.		EENSTMB20

### VITA AKZENT® LC

**Reliable reproduction of individual shade effects**



#### Product description

- VITA AKZENT LC is a light-curing stain and glaze system used for characterizing all indirect restorations made of composite, polymer and hybrid ceramic.
- The VITA AKZENT LC EFFECT STAINS include 14 stains for individual reproduction of effects.
- VITA AKZENT LC CHROMA STAINS are customized stains for the reliable implementation of shade corrections.
- The system also offers a glaze material, brushes, a mixing tray and cleaning liquid.

#### Features/Benefits

- Reliable reproduction of all the shade facets of natural dentition, thanks to a wide range of brilliant effect stains
- Simple realization of shade modifications with a brush stroke, using chroma shades for plannable, systematic control of the chromaticity of dental restorations
- Apply individual shade effects with pinpoint accuracy and apply shades highly efficiently over a large area, thanks to effect and chroma stains with ideally matched viscosities
- Produce restorations with consistently brilliant shade rendering and high gloss stability, thanks to stains with good wear and discoloration resistance

#### Indications

VITA AKZENT LC

- can be used universally for characterizing restorations made of hybrid ceramic, CAD/CAM and veneering composites, as well as acrylic dentures.
- can be used for all indirect restorations made of the above-mentioned materials:  
e.g., CAD/CAM-fabricated crowns, composite veneered bridge restorations, full/partial dentures and long-term temporary restorations
- 

#### Shades

Available in 14 effect stains and four chroma stains

**VITA AKZENT® LC kits****VITA AKZENT® LC****VITA AKZENT® LC EXPERT KIT VITA SYSTEM 3D-MASTER®****Stains for effect reproductions & shade corrections****CALCEKIT3D**

Quantity	Content	Material
6	2	VITA AKZENT LC EFFECT STAINS cream, lemon, sun, pink, purple, black
1		VITA AKZENT LC EFFECT STAINS, dark-red, 2,5 ml
1		VITA AKZENT LC EFFECT STAINS, grey-blue, 2,5 ml
1		VITA AKZENT LC CHROMA STAINS, corn-yellow (L), 2,5 ml
1		VITA AKZENT LC CHROMA STAINS, red-yellow (M2), 2,5 ml
1		VITA AKZENT LC CHROMA STAINS, dark-yellow (M3), 2,5 ml
1		VITA AKZENT LC CHROMA STAINS, light-red (R), 2,5 ml
1	5 ml	VITA AKZENT LC GLAZE
1	25 pcs.	VITA ADIVA Microbrush
1		Brush No. 1 correction brush, red marten
1		Flat brush for CHROMA STAINS

**VITA AKZENT® LC EXPERT KIT VITA classical A1-D4®****Stains for effect reproductions & shade corrections****CALCEKITC**

Quantity	Content	Material
6	2	VITA AKZENT LC EFFECT STAINS cream, lemon, sun, pink, purple, black
1		VITA AKZENT LC EFFECT STAINS, dark-red, 2,5 ml
1		VITA AKZENT LC EFFECT STAINS, grey-blue, 2,5 ml
1		VITA AKZENT LC CHROMA STAINS, red-brown (A), 2,5 ml
1		VITA AKZENT LC CHROMA STAINS, red-sun (B), 2,5 ml
1		VITA AKZENT LC CHROMA STAINS, grey-brown (C), 2,5 ml
1		VITA AKZENT LC CHROMA STAINS, grey-red (D), 2,5 ml
1	5 ml	VITA AKZENT LC GLAZE
1	25 pcs.	VITA ADIVA Microbrush
1		Brush No. 1 correction brush, red marten
1		Flat brush for CHROMA STAINS

**VITA AKZENT® LC STANDARD KIT****Stains for effect reproductions****CALCSKIT**

Quantity	Content	Material
5	2	VITA AKZENT LC EFFECT STAINS white, orange, russet, khaki, blue
1		VITA AKZENT LC EFFECT STAINS, grey-blue, 2,5 ml
1	5 ml	VITA AKZENT LC GLAZE
1	25 pcs.	VITA ADIVA Microbrush
1		Brush No. 1 correction brush, red marten
1		Flat brush for CHROMA STAINS


**VITA AKZENT® LC products****VITA AKZENT® LC**

Description	Prod. no.
VITA AKZENT LC CHROMA STAINS, red-brown (A), 2,5 ml	B5150A25
VITA AKZENT LC CHROMA STAINS, red-sun (B), 2,5 ml	B5150B25
VITA AKZENT LC CHROMA STAINS, grey-brown (C), 2,5 ml	B5150C25
VITA AKZENT LC CHROMA STAINS, grey-red (D), 2,5 ml	B5150D25
VITA AKZENT LC CHROMA STAINS, corn-yellow (L), 2,5 ml	B5150L25
VITA AKZENT LC CHROMA STAINS, red-yellow (M2), 2,5 ml	B5150M225
VITA AKZENT LC CHROMA STAINS, dark-yellow (M3), 2,5 ml	B5150M325
VITA AKZENT LC CHROMA STAINS, light-red (R), 2,5 ml	B5150R25

## VITA AKZENT LC EFFECT STAINS, 2

Description	Prod. no. 2
white	B5155125
cream	B5155225
lemon	B5155325
sun	B5155425
orange	B5155525
russet	B5155625
khaki	B5155725
pink	B5155825
dark-red	B5155925
purple	B5156025
blue	B5156125
grey	B5156325
black	B5156425

Description	Prod. no.
VITA AKZENT LC GLAZE, 5 ml	B515905

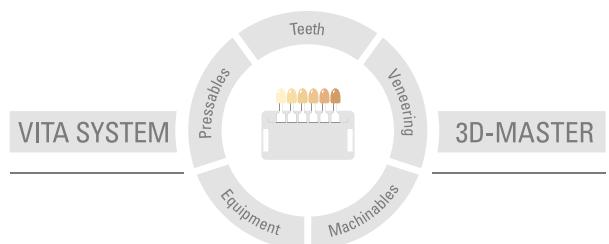

This information issued: 09-2021

Our products were developed specifically for dental use and must be processed according to the instructions for use. The information/illustrations contained herein do not constitute an assurance of properties and are not binding.

# VITA

VITA Zahnfabrik H. Rauter GmbH & Co.KG  
Postfach 1338 · D-79704 Bad Säckingen · Germany  
Tel. +49(0)7761/562-0 · Fax +49(0)7761/562-299  
Hotline: Tel. +49(0)7761/562-222 · Fax +49(0)7761/562-446  
[www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com) · [info@vita-zahnfabrik.com](mailto:info@vita-zahnfabrik.com)  
 [facebook.com/vita.zahnfabrik](https://facebook.com/vita.zahnfabrik)