
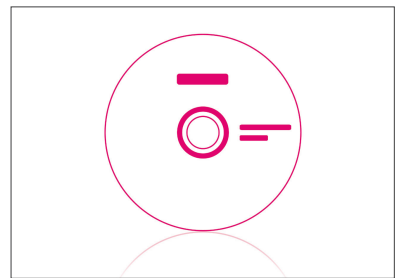
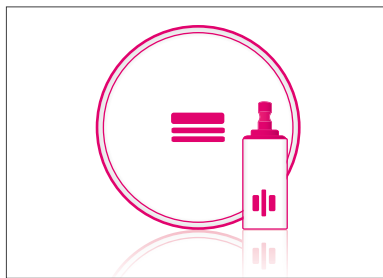
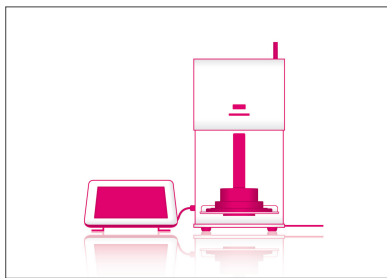
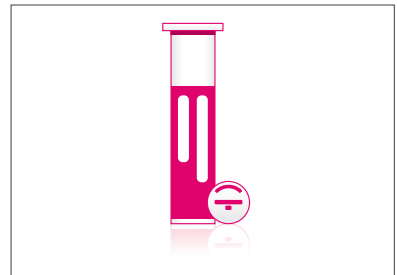



VITA Compendium November 2022

Characterization

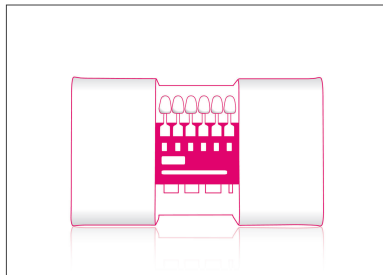

VITA – perfect match.

VITA

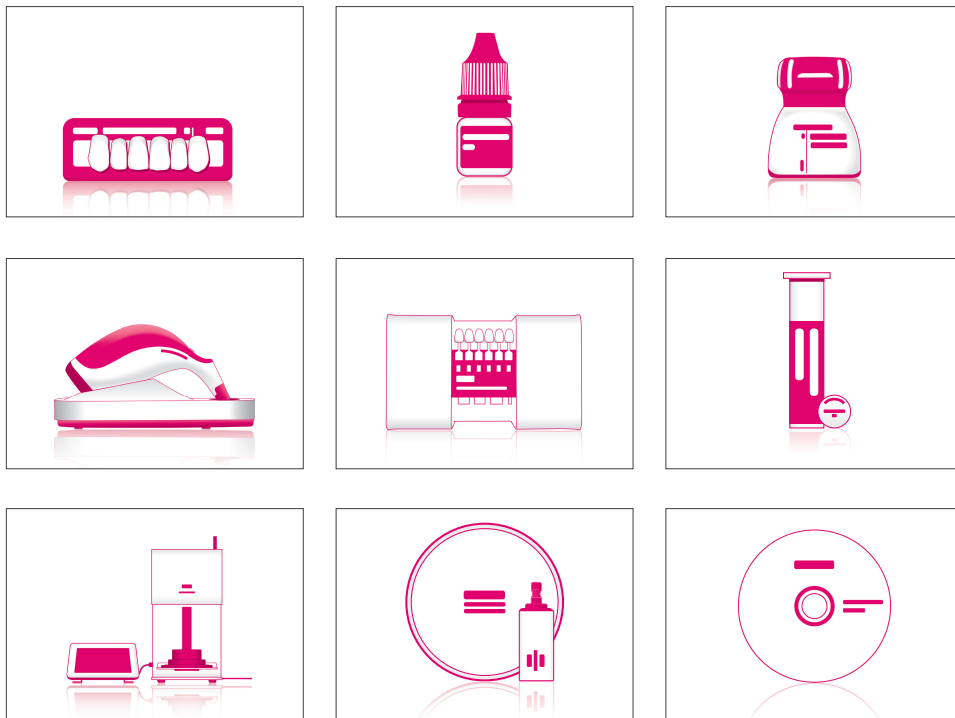

VITA – perfect match.

VITA

VITA AKZENT® Plus

Optimizes everything. Effortlessly.



Product description

New areas of application call for new, versatile materials. With VITA AKZENT Plus stains, users can optimize the shade of all types of dental ceramic restorations simply and efficiently, regardless of the CTE of the restoration.

No matter whether internal coloration, surface characterization or fine glazing: with the new VITA AKZENT Plus fluorescent stains, you can adapt your restoration easily and without undue effort.

Depending on the user's preferred/customary method of processing and the relevant area of application, VITA AKZENT Plus stains are available as powders and as a ready-to-use pastes.

The glazing BODY STAINS and GLAZE material are also available as a spray, which, thanks to even application, ensures uniform surface treatment across a large area without any wastage as a result of scatter. The new, special spray head allows precise application to specific areas.

Features/Benefits

For all types of dental-ceramic materials, for the layering and press technique and feldspar ceramic blocks such as VITABLOCS to monolithic restorations. It is suitable for all dental ceramic materials and for every CTE.

Available in three different forms depending on the method of application:

POWDER:

for unlimited flexibility and cost-effectiveness

PASTE:

ready-to-use pastes with uniform consistency and homogeneous pigmentation

SPRAY:

ready-to-use, easy-to-apply glaze and finishing agent stains

VITA AKZENT® Plus kits

VITA AKZENT® Plus Powder



VITA AKZENT® Plus CHROMA STAINS KIT 3D-MASTER BAPKCS3D

Quantity	Content	Material
5	3 g	VITA AKZENT Plus CHROMA STAINS CHROMA L, CHROMA R, CHROMA IO, CHROMA M2 - CHROMA M3
1	20 ml	VITA AKZENT Plus POWDER FLUID
1		VITA AKZENT Plus CHROMA STAINS, color sample 3D-MASTER



VITA AKZENT® Plus CHROMA STAINS KIT classical BAPKCS

Quantity	Content	Material
4	3 g	VITA AKZENT Plus CHROMA STAINS CHROMA A, CHROMA B, CHROMA C, CHROMA D
1	20 ml	VITA AKZENT Plus POWDER FLUID
1		VITA AKZENT Plus CHROMA STAINS, color sample classical

VITA AKZENT® PLUS MINI KIT BAPKSKV1

Quantity	Content	Material
5	3 g	VITA AKZENT Plus EFFECT STAINS ES01, ES05 - ES06, ES11, ES13
1	5 g	VITA AKZENT Plus GLAZE LT POWDER
1	20 ml	VITA AKZENT Plus POWDER FLUID
1		VITA AKZENT Plus Shade sample disc
1		VITA AKZENT Plus mixing plate, crystal glass (70 x 95 mm)
1		Paint brush no. 3/0 synthetic

VITA AKZENT® Plus POWDER KIT BAPKV1

Quantity	Content	Material
14	3 g	VITA AKZENT Plus EFFECT STAINS ES01 - ES14
5	3 g	VITA AKZENT Plus BODY STAINS BS01 - BS05
1	5 g	VITA AKZENT Plus GLAZE POWDER
1	5 g	VITA AKZENT Plus GLAZE LT POWDER
1	5 g	VITA AKZENT Plus FINISHING AGENT POWDER
1	20 ml	VITA AKZENT Plus POWDER FLUID
1		VITA AKZENT Plus mixing plate incl. shade sample disc
1		VITA glass spatula for stains
1		Measuring spoon, 0,1 ml / Mixing spatula 2-sided
1		Paint brush no. 3/0 synthetic

VITA AKZENT® Plus Paste



VITA AKZENT® Plus CHROMA STAINS PASTE KIT 3D-MASTER BAPKPCS3D

Quantity	Content	Material
5	4 g	VITA AKZENT Plus CHROMA STAINS CHROMA L, CHROMA R, CHROMA IO, CHROMA M2 - CHROMA M3
1	15 ml	VITA AKZENT Plus PASTE FLUID
1		VITA AKZENT Plus CHROMA STAINS, color sample 3D-MASTER



VITA AKZENT® Plus CHROMA STAINS PASTE KIT classical BAPKPCSC

Quantity	Content	Material
4	4 g	VITA AKZENT Plus CHROMA STAINS CHROMA A, CHROMA B, CHROMA C, CHROMA D
1	15 ml	VITA AKZENT Plus PASTE FLUID
1		VITA AKZENT Plus CHROMA STAINS, color sample classical

**VITA AKZENT® Plus PASTE KIT
BAPKPV1**

Quantity	Content	Material
14	4 g	VITA AKZENT Plus EFFECT STAINS ES01 - ES14
5	4 g	VITA AKZENT Plus BODY STAINS BS01 - BS05
1	4 g	VITA AKZENT Plus GLAZE PASTE
1	4 g	VITA AKZENT Plus GLAZE LT PASTE
1	4 g	VITA AKZENT Plus FINISHING AGENT PASTE
1	15 ml	VITA AKZENT Plus PASTE FLUID
1		Flat brush, for PASTE OPAQUE
1		VITA AKZENT Plus Shade sample disc
1		VITA AKZENT Plus mixing plate, crystal glass (70 x 95 mm)
1		VITA glass spatula for stains
1		Paint brush no. 3/0 synthetic

VITA AKZENT® Plus Spray

**VITA AKZENT® Plus STARTER KIT SPRAY / PASTE
BAPSKSM**

Quantity	Content	Material
1	75 ml	VITA AKZENT Plus GLAZE LT Spray
1		Brush No. 1 correction brush, red marten
1		Mixing spatula, Acrylic for PASTE OPAQUE
2	4 g	VITA AKZENT Plus CHROMA STAINS CHROMA A, CHROMA B
2	4 g	VITA AKZENT Plus EFFECT STAINS ES01, ES11
1	4 g	VITA AKZENT Plus GLAZE LT PASTE

VITA AKZENT® Plus products

VITA AKZENT® Plus Powder

VITA AKZENT Plus BODY STAINS, 3 g (Powder)

Description	Prod. no. 3 g (Powder)
BS01	B505013
BS02	B505023
BS03	B505033
BS04	B505043
BS05	B505053

VITA AKZENT Plus CHROMA STAINS, 3 g (Powder)

Description	Prod. no. 3 g (Powder)
CHROMA A	B5050A3
CHROMA B	B5050B3
CHROMA C	B5050C3
CHROMA D	B5050D3
CHROMA IO	B5050IO3
CHROMA L	B5050L3
CHROMA M2	B5050M23
CHROMA M3	B5050M33
CHROMA R	B5050R3

VITA AKZENT Plus EFFECT STAINS, 3 g (Powder)

Description	Prod. no. 3 g (Powder)
ES01	B505513
ES02	B505523
ES03	B505533
ES04	B505543
ES05	B505553
ES06	B505563
ES07	B505573
ES08	B505583
ES09	B505593
ES10	B505603
ES11	B505613
ES12	B505623
ES13	B505633
ES14	B505643

Description	Prod. no.
VITA AKZENT Plus GLAZE POWDER, 30 g	B5058130
VITA AKZENT Plus GLAZE POWDER, 5 g	B505815
VITA AKZENT Plus GLAZE LT POWDER, 5 g	B505825
VITA AKZENT Plus FINISHING AGENT POWDER, 5 g	B505835
VITA AKZENT Plus POWDER FLUID, 20 ml	BAPF20

VITA AKZENT® Plus Paste
VITA AKZENT Plus BODY STAINS, 4 g (Paste)

Description	Prod. no. 4 g (Paste)
BS01	B505014
BS02	B505024
BS03	B505034
BS04	B505044
BS05	B505054

VITA AKZENT Plus CHROMA STAINS, 4 g (Paste)

Description	Prod. no. 4 g (Paste)
CHROMA A	B5050A4
CHROMA B	B5050B4
CHROMA C	B5050C4
CHROMA D	B5050D4
CHROMA IO	B5050IO4
CHROMA L	B5050L4
CHROMA M2	B5050M24
CHROMA M3	B5050M34
CHROMA R	B5050R4

VITA AKZENT Plus EFFECT STAINS, 4 g (Paste)

Description	Prod. no. 4 g (Paste)
ES01	B505514
ES02	B505524
ES03	B505534
ES04	B505544
ES05	B505554
ES06	B505564
ES07	B505574
ES08	B505584
ES09	B505594
ES10	B505604
ES11	B505614
ES12	B505624
ES13	B505634
ES14	B505644

Description	Prod. no.
VITA AKZENT Plus GLAZE PASTE, 4 g	B505814
VITA AKZENT Plus GLAZE LT PASTE, 4 g	B505824
VITA AKZENT Plus FINISHING AGENT PASTE, 4 g	B505834
VITA AKZENT Plus PASTE FLUID, 15 ml	BAPPF15

VITA AKZENT® Plus Spray

Description	Prod. no.
VITA AKZENT Plus GLAZE SPRAY, 75 ml	B5051075
VITA AKZENT Plus GLAZE LT Spray, 75 ml	B5052075
VITA AKZENT Plus FLUOGLAZE LT, 75 ml	B5053075

VITA INTERNO®

For reproducing the shade effects found in the tooth interior



Product description

The VITA INTERNO ceramics enable the ceramist to achieve an even more perfect reproduction of in-depth shade effects. Outstanding results can be obtained with INTERNO ceramics, particularly when reproducing colour effects (contrasts) occurring in the incisal area of natural teeth. In special cases the INTERNO ceramics can be used to create an additional impression of depth when space is lacking. Their high fluorescence means shades with increased luminosity and even better light transmission - just as in natural dentition.

VITA INTERNO powders are compatible with **all** VITA veneering ceramics.

Features/Benefits

- Compatible with all VITA veneering ceramics
- Can be mixed together
- Stain shades are based on shades naturally found on teeth, so complicated mixing is usually unnecessary
- Do not run; colorfast

VITA INTERNO® kits

VITA INTERNO®

**VITA INTERNO KIT
BISETV3**

Quantity	Content	Material
14	3 g	VITA INTERNO Int01 - Int14
1	20 ml	VITA INTERNO FLUID
1		VITA INTERNO Shade sample disc
1		VITA INTERNO mixing palette, acrylic (Ø 144 mm, with indentations)
1		VITA glass spatula for stains
1		Paint brush no. 3/0 synthetic

VITA INTERNO® products

VITA INTERNO®

VITA INTERNO, 3 g

Description	Prod. no. 3 g
Int01	BI013
Int02	BI023
Int03	BI033
Int04	BI043
Int05	BI053
Int06	BI063
Int07	BI073
Int08	BI083
Int09	BI093
Int10	BI103
Int11	BI113
Int12	BI123
Int13	BI133
Int14	BI143

Description	Prod. no.
VITA INTERNO FLUID, 20 ml	BIF20

VITA AKZENT® LC

Reliable reproduction of individual shade effects



Product description

- VITA AKZENT LC is a light-curing stain and glaze system used for characterizing all indirect restorations made of composite, polymer and hybrid ceramic.
- The VITA AKZENT LC EFFECT STAINS include 14 stains for individual reproduction of effects.
- VITA AKZENT LC CHROMA STAINS are customized stains for the reliable implementation of shade corrections.
- The system also offers a glaze material, brushes, a mixing tray and cleaning liquid.

Features/Benefits

- Reliable reproduction of all the shade facets of natural dentition, thanks to a wide range of brilliant effect stains
- Simple realization of shade modifications with a brush stroke, using chroma shades for plannable, systematic control of the chromaticity of dental restorations
- Apply individual shade effects with pinpoint accuracy and apply shades highly efficiently over a large area, thanks to effect and chroma stains with ideally matched viscosities
- Produce restorations with consistently brilliant shade rendering and high gloss stability, thanks to stains with good wear and discoloration resistance

Indications

VITA AKZENT LC

- can be used universally for characterizing restorations made of hybrid ceramic, CAD/CAM and veneering composites, 3D-print acrylic polymers, as well as acrylic dentures.
- can be used for all indirect restorations made of the above-mentioned materials:
e.g., CAD/CAM-fabricated crowns, composite veneered bridge restorations, full/partial dentures and long-term temporary restorations
-

Shades

Available in 14 effect stains and four chroma stains

VITA AKZENT® LC kits

VITA AKZENT® LC

VITA AKZENT® LC EXPERT KIT VITA SYSTEM 3D-MASTER®

Stains for effect reproductions & shade corrections

CALCEKIT3D

Quantity	Content	Material
8	2.5 ml	VITA AKZENT LC EFFECT STAINS cream, lemon, sun, pink, dark-red, purple, grey, black
1	2.5 ml	VITA AKZENT LC CHROMA STAINS, corn-yellow (L)
1	2.5 ml	VITA AKZENT LC CHROMA STAINS, red-yellow (M2)
1	2.5 ml	VITA AKZENT LC CHROMA STAINS, dark-yellow (M3)
1	2.5 ml	VITA AKZENT LC CHROMA STAINS, light-red (R)
1	5 ml	VITA AKZENT LC GLAZE
1	25 ml	VITA AKZENT LC CLEANER
1		VITA Mixing palette white porcelain with lid, 5,5 x 3,5 cm
1	25 pcs.	VITA ADIVA Microbrush
1		Brush No. 1 correction brush, red marten
1		Flat brush for CHROMA STAINS

VITA AKZENT® LC EXPERT KIT VITA classical A1-D4®

Stains for effect reproductions & shade corrections

CALCEKITC

Quantity	Content	Material
8	2.5 ml	VITA AKZENT LC EFFECT STAINS cream, lemon, sun, pink, dark-red, purple, grey, black
1	2.5 ml	VITA AKZENT LC CHROMA STAINS, red-brown (A)
1	2.5 ml	VITA AKZENT LC CHROMA STAINS, red-sun (B)
1	2.5 ml	VITA AKZENT LC CHROMA STAINS, grey-brown (C)
1	2.5 ml	VITA AKZENT LC CHROMA STAINS, grey-red (D)
1	5 ml	VITA AKZENT LC GLAZE
1	25 ml	VITA AKZENT LC CLEANER
1		VITA Mixing palette white porcelain with lid, 5,5 x 3,5 cm
1	25 pcs.	VITA ADIVA Microbrush
1		Brush No. 1 correction brush, red marten
1		Flat brush for CHROMA STAINS

VITA AKZENT® LC STANDARD KIT

Stains for effect reproductions


CALCSKIT

Quantity	Content	Material
6	2.5 ml	VITA AKZENT LC EFFECT STAINS white, orange, russet, khaki, blue, grey-blue
1	5 ml	VITA AKZENT LC GLAZE
1	25 ml	VITA AKZENT LC CLEANER
1		VITA Mixing palette white porcelain with lid, 5,5 x 3,5 cm
1	25 pcs.	VITA ADIVA Microbrush
1		Brush No. 1 correction brush, red marten
1		Flat brush for CHROMA STAINS

VITA AKZENT® LC products

VITA AKZENT® LC

Description	Prod. no.
VITA AKZENT LC CHROMA STAINS, red-brown (A), 2.5 ml	B5150A25
VITA AKZENT LC CHROMA STAINS, red-sun (B), 2.5 ml	B5150B25
VITA AKZENT LC CHROMA STAINS, grey-brown (C), 2.5 ml	B5150C25
VITA AKZENT LC CHROMA STAINS, grey-red (D), 2.5 ml	B5150D25
VITA AKZENT LC CHROMA STAINS, corn-yellow (L), 2.5 ml	B5150L25
VITA AKZENT LC CHROMA STAINS, red-yellow (M2), 2.5 ml	B5150M225
VITA AKZENT LC CHROMA STAINS, dark-yellow (M3), 2.5 ml	B5150M325
VITA AKZENT LC CHROMA STAINS, light-red (R), 2.5 ml	B5150R25
VITA AKZENT LC EFFECT STAINS, white, 2.5 ml	B5155125
VITA AKZENT LC EFFECT STAINS, cream, 2.5 ml	B5155225
VITA AKZENT LC EFFECT STAINS, lemon, 2.5 ml	B5155325
VITA AKZENT LC EFFECT STAINS, sun, 2.5 ml	B5155425
VITA AKZENT LC EFFECT STAINS, orange, 2.5 ml	B5155525
VITA AKZENT LC EFFECT STAINS, russet, 2.5 ml	B5155625
VITA AKZENT LC EFFECT STAINS, khaki, 2.5 ml	B5155725
VITA AKZENT LC EFFECT STAINS, pink, 2.5 ml	B5155825
VITA AKZENT LC EFFECT STAINS, dark-red, 2.5 ml	B5155925
VITA AKZENT LC EFFECT STAINS, purple, 2.5 ml	B5156025
VITA AKZENT LC EFFECT STAINS, blue, 2.5 ml	B5156125
VITA AKZENT LC EFFECT STAINS, grey-blue, 2.5 ml	B5156225
VITA AKZENT LC EFFECT STAINS, grey, 2.5 ml	B5156325
VITA AKZENT LC EFFECT STAINS, black, 2.5 ml	B5156425
VITA AKZENT LC GLAZE, 5 ml	B515905

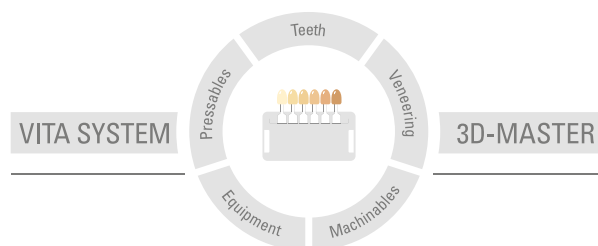

This information issued: 11-2022

Our products were developed specifically for dental use and must be processed according to the instructions for use. The information/illustrations contained herein do not constitute an assurance of properties and are not binding.

VITA

VITA Zahnfabrik H. Rauter GmbH & Co.KG
Postfach 1338 · D-79704 Bad Säckingen · Germany
Tel. +49 (0) 7761/562-0 · Fax +49 (0) 7761/562-299
Hotline: Tel. +49 (0) 7761/562-222 · Fax +49 (0) 7761/562-446
www.vita-zahnfabrik.com · info@vita-zahnfabrik.com
 facebook.com/vita.zahnfabrik