
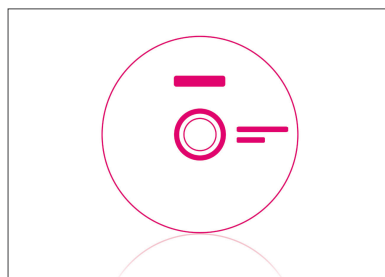
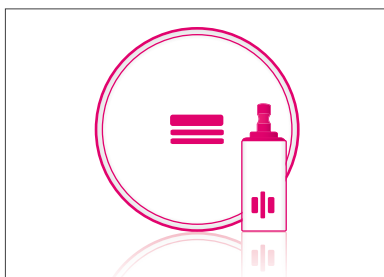
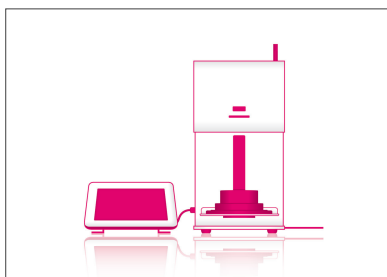
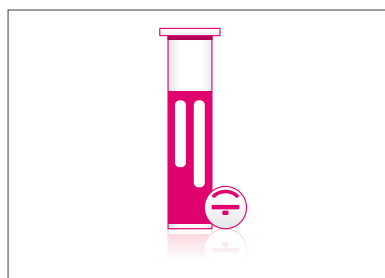



# VITA Compendium November 2023

## Polishing

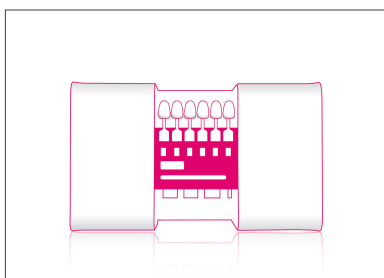

VITA – perfect match.

**VITA**

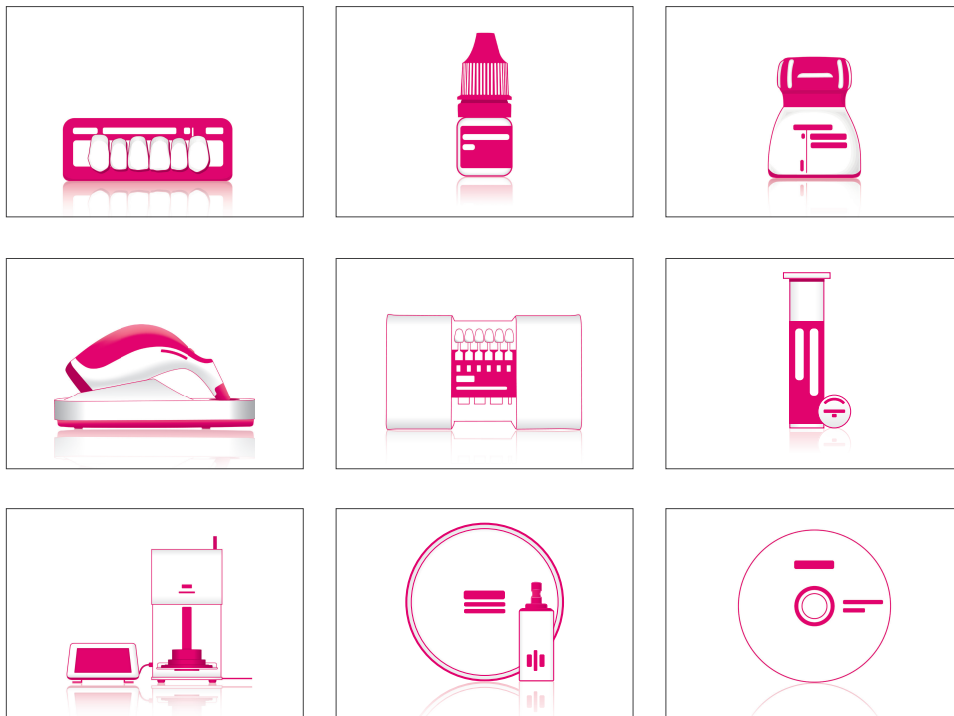

VITA – perfect match.

**VITA**

## VITA CERAMICS Polishing Set

For the pre- and high-gloss polishing of all ceramic materials



### Product description

VITA CERAMICS Polishing Sets were developed especially for the two-step polishing of all dental ceramic materials in practices and laboratories. The sets each contain a total of six polishing instruments for pre- and high-gloss polishing.

### Features/Benefits

#### For all ceramic:

**Feldspar ceramic, glass ceramic, oxide ceramic:** All types of ceramic can be polished with just one set.

#### Simple polishing, excellent performance:

With their excellent removal and polishing performance, the instruments are impressive, even in the case of pronounced fissures. Safe use of these clinical instruments is also guaranteed through sterilization.

#### Long use:

VITA CERAMICS Polishing Sets are particularly efficient and durable, due to a high proportion of diamonds and a large usable volume.

### Content

#### VITA CERAMICS Polishing Set clinical:

Includes a total of six instruments for the contra-angle. Includes three instruments each for pre- and high-gloss polishing. All instruments can be sterilized.

#### VITA CERAMICS Polishing Set technical:


Includes a total of six instruments for the handpiece. Includes three instruments each for pre- and high-gloss polishing.

**VITA CERAMICS Polishing Set products****VITA CERAMICS Polishing Set**

Description	Prod. no.
VITA CERAMICS Polishing Set clinical	ECEPSETC
VITA CERAMICS Polishing Set technical	ECEPSETT
Polishing instrument technical, lens, VI-H16D high-gloss polishing (cream), 3 pcs.	ERHH16D3
Polishing instrument technical, lens, VI-H16Dmf pre-polishing (grey), 3 pcs.	ERHH16DMF3
Polishing instrument technical, Brush, VI-DT-H17D high-gloss polishing (cream), 3 pcs.	ERHH17D3
Polishing instrument technical, Brush, VI-DT-H17Dmf pre-polishing (grey), 3 pcs.	ERHH17DMF3
Polishing instrument technical, tip, VI-H1D high-gloss polishing (cream), 6 pcs.	ERHH1D6
Polishing instrument technical, tip, VI-H1Dmf pre-polishing (grey), 6 pcs.	ERHH1DMF6
Polishing instrument clinical, Brush, VI-DT-W14D high-gloss polishing (cream), 3 pcs.	ERWW14D3
Polishing instrument clinical, Brush, VI-DT-W14Dmf pre-polishing (grey), 3 pcs.	ERWW14DMF3
Polishing instrument clinical, lens, VI-W18D high-gloss polishing (cream), 6 pcs.	ERWW18D6
Polishing instrument clinical, lens, VI-W18Dmf pre-polishing (grey), 6 pcs.	ERWW18DMF6
Polishing instrument clinical, cup, VI-W7D high-gloss polishing (cream), 6 pcs.	ERWW7D6
Polishing instrument clinical, cup, VI-W7Dmf pre-polishing (grey), 6 pcs.	ERWW7DMF6

## VITA ENAMIC® Polishing Set

For simple pre- and high-gloss polishing of VITA ENAMIC hybrid ceramic

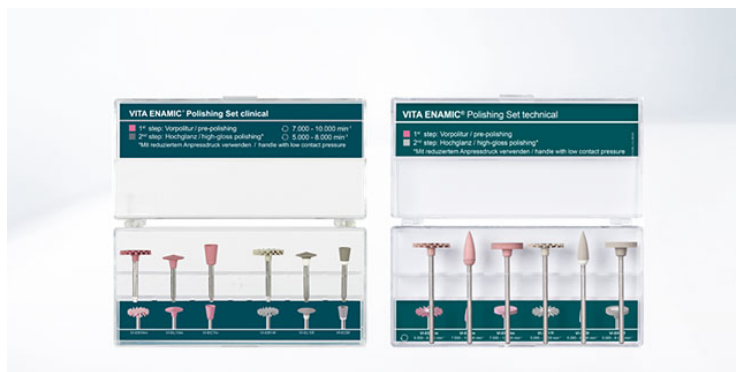

### Product description

The VITA ENAMIC polishing sets were developed for reliable, efficient and material-specific surface treatment of hybrid ceramic restorations in dental practices and laboratories. The sets include various polishing instruments for pre- and high-gloss polishing. These instruments are suitable for careful and gentle polishing of occlusal surfaces, cusps, fissures and contact points of the restoration and produce surfaces with exceptional gloss.

### Features/Benefits

#### Excellent final results:

High Gloss and plaque-resistant surfaces can be produced with these instruments. Precise concentricity, matched grit sizes and the individual geometries of the instruments guarantee results with superior precision.

#### Simple and safe handling:

The instruments provide superior material removal performance. Good handling and the use without polishing paste enable simple and fast processing.

#### Gentle and careful processing:

These instruments were developed specifically for the VITA ENAMIC to ensure gentle and careful polishing, reducing the risk of possible microcrack formation.

### Content



#### VITA ENAMIC Polishing Set clinical:

Includes a total of 6 instruments for the contra-angle; 3 instruments for pre-polishing and 3 instruments for high-gloss polishing.

#### VITA ENAMIC Polishing Set technical:

Includes a total of 6 instruments for the handpiece; 3 instruments for pre-polishing and 3 instruments for high-gloss polishing.

**VITA ENAMIC® Polishing Set products****VITA ENAMIC® Polishing Set**

Description		Prod. no.
VITA ENAMIC Polishing Set clinical		EENPSETCV1
VITA ENAMIC Polishing Set technical		EENPSETTV1
Polishing instrument technical, Brush, VI-EB17f high-gloss polishing (grey), 3 pcs.		ERHEB17F3
Polishing instrument technical, Brush, VI-EB17m pre-polishing (pink), 3 pcs.		ERHEB17M3
Polishing instrument technical, wheel, VI-ER15f high-gloss polishing (grey), 6 pcs.		ERHER15F6
Polishing instrument technical, wheel, VI-ER15m pre-polishing (pink), 6 pcs.		ERHER15M6
Polishing instrument technical, tip, VI-ES5f high-gloss polishing (grey), 6 pcs.		ERHES5F6
Polishing instrument technical, tip, VI-ES5m pre-polishing (pink), 6 pcs.		ERHES5M6
Polishing instrument clinical, Brush, VI-EB14f high-gloss polishing (grey), 3 pcs.		ERWEB14F3
Polishing instrument clinical, Brush, VI-EB14m pre-polishing (pink), 3 pcs.		ERWEB14M3
Polishing instrument clinical, cup, VI-EC6f high-gloss polishing (grey), 6 pcs.		ERWEC6F6
Polishing instrument clinical, cup, VI-EC7m pre-polishing (pink), 6 pcs.		ERWEC7M6
Polishing instrument clinical, lens, VI-EL10f high-gloss polishing (grey), 6 pcs.		ERWEL10F6
Polishing instrument clinical, lens, VI-EL10m pre-polishing (pink), 6 pcs.		ERWEL10M6

## VITA SUPRINITY® Polishing Set

For simple pre- and high-gloss polishing of VITA SUPRINITY PC



### Product description

The VITA SUPRINITY Polishing Sets were developed for reliable, efficient and material-specific surface treatment of zirconia reinforced lithium silicate ceramic (ZLS) restorations in dental practices and laboratories. The sets include various polishing instruments for pre- and high-gloss polishing.

These instruments are suitable for careful and gentle polishing of occlusal surfaces, cusps, fissures and restoration contact points. They ensure a brilliant shine on the finished restoration.

### Features/Benefits

#### Excellent final results:

The instruments produce highly esthetic and plaque-resistant surfaces. Precise concentricity, matched grit sizes and the individual geometries of the instruments guarantee superior results.

#### Simple and safe handling:

The instruments provide removal capacity that can be easily controlled with minimal wear. Good handling and the ability to use them without polishing paste enables simple and fast processing. Safe use is also guaranteed since the instruments can be sterilized.

#### Gentle and careful processing:

These instruments, which were developed especially for VITA SUPRINITY PC, ensure gentle and careful processing. As a result, the risk of possible formation of microcracks is reduced.

### Content



#### VITA SUPRINITY Polishing Set clinical:

Includes a total of 6 instruments for the contra-angle; 3 instruments for pre-polishing and 3 instruments for high-gloss polishing.

#### VITA SUPRINITY Polishing Set technical:

Includes a total of 8 instruments for the handpiece; 4 instruments for pre-polishing and 4 instruments for high-gloss polishing.

**VITA SUPRINITY® Polishing Set products****VITA SUPRINITY® Polishing Set**

Description		Prod. no.
VITA SUPRINITY Polishing Set clinical		ELSPSETC
VITA SUPRINITY Polishing Set technical		ELSPSETT
Polishing instrument technical, lens, L16f high-gloss polishing (grey), 6 pcs.		ERHL16F6
Polishing instrument technical, lens, L16m pre-polishing (pink), 6 pcs.		ERHL16M6
Polishing instrument technical, long pin, LPf high-gloss polishing (grey), 6 pcs.		ERHLPF6
Polishing instrument technical, long pin, LPm pre-polishing (pink), 6 pcs.		ERHLPm6
Polishing instrument technical, wheel, R15f high-gloss polishing (grey), 6 pcs.		ERHR15F6
Polishing instrument technical, wheel, R15m pre-polishing (pink), 6 pcs.		ERHR15M6
Polishing instrument technical, tip, S5f high-gloss polishing (grey), 6 pcs.		ERHS5F6
Polishing instrument technical, tip, S5m pre-polishing (pink), 6 pcs.		ERHS5M6
Polishing instrument clinical, cup, C6f high-gloss polishing (grey), 6 pcs.		ERWC6F6
Polishing instrument clinical, cup, C6m pre-polishing (pink), 6 pcs.		ERWC6M6
Polishing instrument clinical, lens, L10f high-gloss polishing (grey), 6 pcs.		ERWL10F6
Polishing instrument clinical, lens, L10m pre-polishing (pink), 6 pcs.		ERWL10M6
Polishing instrument clinical, tip, S4f high-gloss polishing (grey), 6 pcs.		ERWS4F6
Polishing instrument clinical, tip, S4m pre-polishing (pink), 6 pcs.		ERWS4M6



## VITA polishing pastes

**Diamond polishing pastes for extraoral high-gloss polishing of ceramic and polymer containing materials**



### Product description

**VITA Polish Hybrid** is a diamond polishing paste for extraoral and final high-gloss polishing of restorations made of VITA ENAMIC hybrid ceramic, composite and polymer.

**VITA Polish Cera** is a diamond polishing paste for extraoral and final high-gloss polishing of restorations made of dental ceramic materials: feldspar ceramics, glass ceramics and zirconia ceramics.

### Features/Benefits

- Both polishing pastes enable quick and reliable high-gloss results. At the same time, the surface texture is preserved.
- The diamond particles are each specifically matched to the material classes. As a result, this improves the surface hardness and reduces the likelihood of plaque accumulation.



### Content


VITA Polish Hybrid polishing paste is very suitable in connection with the polishing of VITA ENAMIC hybrid ceramic, in conjunction with the polishing instruments of the VITA ENAMIC Polishing Sets clinical and technical.

VITA Polish Cera polishing paste is very suitable in connection with the polishing of VITA SUPRINITY PC, VITA AMBRIA and VITA YZ, in conjunction with the polishers of the VITA SUPRINITY Polishing Sets clinical and technical.

**VITA polishing pastes products**

**VITA polishing pastes**

Description		Prod. no.
VITA Polish Cera, 18 g		B459
VITA Polish Hybrid, 12 g		C020

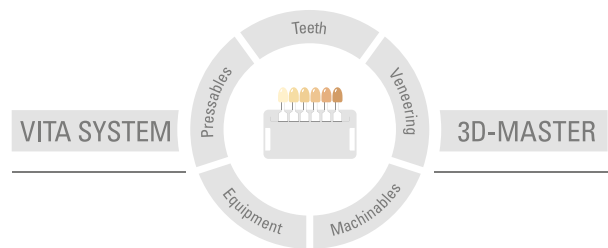

This information issued: 11-2023

Our products were developed specifically for dental use and must be processed according to the instructions for use. The information/illustrations contained herein do not constitute an assurance of properties and are not binding.

# VITA

VITA Zahnfabrik H. Rauter GmbH & Co.KG  
Postfach 1338 · D-79704 Bad Säckingen · Germany  
Tel. +49 (0) 7761/562-0 · Fax +49 (0) 7761/562-299  
Hotline: Tel. +49 (0) 7761/562-222 · Fax +49 (0) 7761/562-446  
[www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com) · [info@vita-zahnfabrik.com](mailto:info@vita-zahnfabrik.com)  
 [facebook.com/vita.zahnfabrik](https://facebook.com/vita.zahnfabrik)